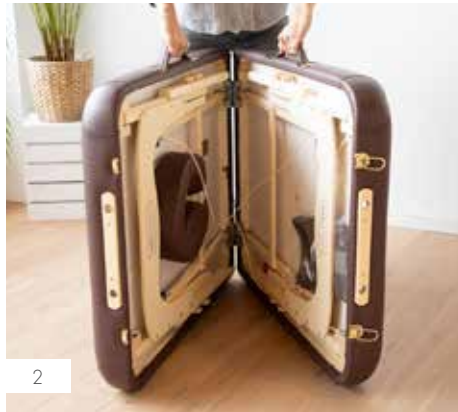
EN | Opening the table: Place the folded table on the rubber bumpers with the handles facing up. Undo the latches (Fig. 1) and open the table (Fig. 2) until the legs are fully extended, making sure that the steel cables do not become entangled (Fig. 3). | **Setting the table upright:** When the table is fully opened, tilt the table to an upright position until all four feet are on the floor. It can help to immobilise one leg with your foot when doing this (Fig. 4). Lift one end of the table and press down on the middle to apply a little tension to the cables. (Fig. 5) | **Changing the table height:** Work on only one leg at a time. Unscrew the knob and remove the bolt. Select the hole for the desired height and put the bolt through that hole. Tighten the knob by hand until snug. Don't over tighten. Repeat at the same height on all other legs (Fig. 6). | **Closing the table:** Turn the table on its side with the rubber bumpers down (Fig. 7). Store the facerest under the table (Fig. 8). Fold the legs into the table (Fig. 9), make sure the cables are inside the table before closing the table and closing the latches.

Instructions for care of polyurethane: Your massage table is covered with high-quality polyurethane (PU). It can be easily cleaned with water and a small amount of dishwashing liquid. For occasional disinfection, we recommend our alcohol-free disinfectant cleaner, „Novadest“. Novadest contains no alcohol, aldehydes or phenols, making it very gentle on materials. Simply spray on, leave for 1 minute, then wipe off with a damp soft cloth. Common household bleach, diluted 1:10 with water, is also suitable for cleaning the PU covers. Oils, including the natural oils from your skin, can damage the cover material. Therefore, always protect the surface with a clean cover and ensure that no oil residues remain on the material. When using larger amounts of oil, such as in ayurvedic treatments, the table must always be protected with a special oil-resistant cover. **Please note:** Aggressive or alcohol-based disinfectants can also damage the material. Damage caused by the use of cleaning agents other than those specified is not covered by the warranty.

SAFETY INSTRUCTIONS: Check all screw connections and the steel cables annually. Contact us in the event of safety-relevant signs of wear. Unauthorised modifications to the product will void the warranty.

INSTRUCTIONS FOR USE | GEBRAUCHSANLEITUNG | MODE D'EMPLOI | ISTRUZIONI PER L'USO

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